

HUMANE-TO-SLEEP INC. PRESENTS



THE KATY-ONE SYSTEM: THE MOST
HUMANE METHOD FOR FERAL ANIMAL
EUTHANASIA BY USING
COMMERICALLY COMPRESSED CARBON
MONOXIDE

Feral [L.fira a wild animal] Untamed;
undomesticated;
hence, wild; savage.

The KATY-ONE SYSTEM is in compliance with the AVMA who require a 6% to 8% concentration of compressed CO via cylinder to be introduced within 45 seconds.

“When the restraint necessary for giving an animal an intravenous injection would impart added distress to the animal or pose undue risk to the operator ...an acceptable alternate route should be employed. (emphasis mine.)

--Report of the AVMA Panel on Euthanasia
JAVMA Vol.218, No. 5, March 1, 200

ENVIRONMENTAL PROBLEMS: SODIUM PENTABARBITAL versus CARBON MONOXIDE

Sodium pentobarbital is a deadly nerve toxin. Accidental wildlife deaths have been reported when scavengers feed on carcasses injected with sodium pentobarbital.

Also, many animals killed by lethal injection find their way to landfills. Landfill liners can leak. There is a real possibility of contamination, ongoing, to our water tables with this toxin.

As concerned professionals, it is imperative that we use sodium pentobarbital wisely and limit its use when possible to the pet population.

Lethal injection is the system of choice when putting PETS to sleep.

A pet is an animal which has an owner.

Carbon monoxide comes from many sources: car engines, lawnmowers, water heaters, gas dryers and so on. Venting CO to outside air protects your workers. (Remember that while the AVMA requires a 6 percent concentration of CO, many heavy smokers have daily CO blood levels of 9 percent!)

STAFF SAFETY AND THE KATY-ONE SYSTEM

The number one reason for the creation of the Katy-One System was staff safety. Bites, scratches, infections and rabies are true concerns when trying to access a vein on a wild, resisting animal.

OUR UNIQUE ADVANTAGES WITH FERAL ANIMALS

1. Your staff is protected from injuries associated with lethal injection.
2. Struggling cats can be left in carriers.
3. Large and aggressive dogs are managed by a one- or two-person pole system that lead the animal into the cage.
4. Injured, unconscious and contaminated animals can be placed on a litter and wheeled into the chamber.
5. Your staff is not exposed to rabies.
6. Because CO cannot be detected by the animals, there is no panic or stress to them when they are introduced to the chamber.
7. The Katy-One System's multiple CO monitors, computerized exhaust system and safety features ensure the safety of your staff.
 - A. The gas and its flow is controlled by a pre-set computer.
 - B. The gas cannot come on until the door is locked.
 - C. The door cannot be re-opened until the chamber is empty of gas.
 - D. Two keys are required to use the system.
It has a built-in CO detector.

8. When done to the AVMA standards of a CO concentration of 6% accomplished in 45 seconds, the average animal is down in 60 seconds. They feel nothing; there is no discomfort to the animal. This gives a great reduction of stress to your staff, knowing they won't have to witness painful deaths.
9. Your staff can come to work knowing they will be safe.

NOTE: There is a condition known as “vocalizing” that occurs only in dogs. Vocalizing is the dog equivalent to snoring. You have all seen sleeping dogs that whine, woof and paddle when comfortably asleep. It only happens in a state of sedation. It is not a sign of air hunger or distress. Hearing this is your assurance that the animal is not suffering.

COST ADVANTAGES AND FINANCIAL SAVING OF THE KATY-ONE SYSTEM

COST OF OPERATIONS: The amount of gaseous CO used in the chamber is the same every time whether you euthanize twelve cats or one dog. We recommend putting several smaller animals in the large chamber at the same time. This denning instinct is particularly beneficial when working with cats, which are calmer in the presence of other cats, and up to a dozen cats can be placed in the chamber together.

An average tank of CO will allow approximately 151 cycles.

| | |
|----------------|------------------|
| Large chamber: | \$1.05 per cycle |
| Small chamber: | \$0.35 per cycle |
| Average: | \$0.70 per cycle |

1. **SAVINGS ON:**

- A. Euthanasia solutions
- B. Tranquilizers
- C. Needles and syringes

2. STAFF SAVINGS

- D. Reduced time spent on paper work.
No need to track items. (A,B,C)
- E. Save on training fee for each staff member who needs to be trained to perform lethal injection.
- F. This product has the potential to save thousands of dollars in Workman's Compensation claims.
- G. There is no need for a veterinarian to euthanize animals.
The doctors can do what they were trained to do: evaluate and save.
- H. Reduce the theft of scheduled drugs that could be a huge liability.

To bring this point home. Imagine putting down a dozen feral cats at one time, and at any time with no other help for one dollar the largest dog.

For more information contact;
HUMANE TO-SLEEP INC.
Fax: 814-796-0023
Email: jackz@velocity.net